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# FLAEM®

Fabbricante / Manufacturer:  
**FLAEMNUOVA®**

Elettromedicali / Eletrodomestici  
Via Colli Storici, 221, 223, 225 - 25010 S. MARTINO D/B. (BS) - ITALIA  
Tel. +39 030 9910168 r.a. + Fax +39 030 9910287  
www.flaemnuova.it

I

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**GB** The warranty terms here specified are valid only in Italy for Italian residents. In all other countries, the warranty will be provided by the local dealer that sold you the unit, in accordance with the applicable laws.

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**GB**

- |  |                              |
|--|------------------------------|
| 1. SUCTION UNIT                              | 9. ANTIBACTERIAL FILTER      |
| 2. EXTERNAL POWER (EXT. POWER/CHARGE)        | 10. TUBE L. 160              |
| 3. SWITCH                                    | 11. PROTECTION DEVICE        |
| 4. VACUUM REGULATION KNOB (VACUUM REGULATOR) | 12. LID WITH PORTS           |
| 5. VACUUM GAUGE                              | 13. "VACUUM" PORT            |
| 6A. GREEN LED (PUMP ON)                      | 14. "PATIENT" PORT           |
| 6B. RED LED (LOW BATTERY)                    | 15. TUBE L. 1300             |
| 6C. YELLOW LED (CHARGING)                    | 16. COLLECTION JAR           |
| 6D. GREEN LED (EXT. POWER)                   | 17. MANUAL SUCTION REGULATOR |
| 7. AIR INLET PORT (INLET)                    | 18. SUCTION CATHETER         |
| 8. TUBE L. 80                                |                              |

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# Portable professional suction unit for people who have undergone tracheostomy

## Port A Suction

*Congratulations on your purchase and thank you for choosing our unit.*

The **Port A Suction** aspirator (1) is a portable professional unit for home use that is specifically designed for patients who have undergone permanent tracheotomy. The unit is equipped with a vacuum regulation knob (4), a vacuum gauge (5) and a 1000 ml collection jar (16) with a protection device preventing entrance of liquid in the suction pump through a float shut-off located in the collection jar lid. This unit is handy, easy to use, reliable, resistant and silent and does not need lubrication. This model features a rechargeable battery and can also operate in cars or boats through the cigarette-lighter socket. The **Port A Suction** aspirator is supplied with the following FLAEM accessories: 1000 ml collection jar (16) with float shut-off protection device (11), cigarette-lighter cord, multivoltage power unit, connection tubes (8/10/15), **disposable** suction catheter for aspirator (18), **disposable** manual suction regulator (17) and **disposable** antibacterial filter (9). **N.B.:** Use only **FLAEM original accessories**

### IMPORTANT SAFEGUARDS

**This is a medical device. It is to be used only upon medical prescription and must be operated in strict accordance with this manual. It is important that the patient reads and understands the information provided for use and care of the unit. Please contact your dealer with any questions you may have.**

The manufacturer takes every step necessary to ensure utmost product quality and safety. However, as for all electrical appliances, some basic safety rules should be followed even if the unit operates at very low voltage.

- Children and disabled persons should always use the unit under strict supervision of an adult who has read this manual.
- Never use adaptors for power supplies different from the one indicated on the bottom of the unit. Keep the cord away from heated surfaces.
- The unit should not be used in the presence of anaesthetic substances inflammable with air, oxygen or nitrous oxide.
-  Do not handle the power supply unit with wet hands. Never use the unit while having a bath or shower. Never leave the unit close to water, never immerse it in any liquid, do not wet. Should it fall into water, unplug immediately before reaching for the unit. Do not operate the unit if the cord or plug is damaged or wet (return it immediately to an authorised service dealer).
- Unit with case not protected against ingress of liquids.
- Servicing and/or maintenance may only be carried out by authorised personnel. Unauthorised repair voids the warranty.
- Make sure that all connections are properly made and that the jar lid is properly closed in order to prevent suction leaks.
- Be careful not to knock the collection jar over while connected to the operating unit as liquid might be aspirated back into the unit and damage the pump. Should this occur, immediately stop the aspirator. The collection jar should immediately be emptied and cleaned. (Return it to an authorised service dealer).
- The float shut-off protection device (11) stops product suction; empty the collection jar (16) and clean it.
- The suction catheter and the manual suction regulator are sterile and **disposable**; they should be replaced after each application.
- Ensure the cannula is used prior to the expiry date and check that the sterile package of the cannula has not been damaged. **FLAEM NUOVA may not be held responsible for any injuries arising from the incorrect use of this product or improper handling of the product by third parties or if used when the packaging has been damaged. DO NOT USE IF STERILITY HAS BEEN COMPROMISED.**
- The **disposable** antibacterial filter should be replaced within 24 hours after its first use.

- 12V DC power cord for cigarette-lighter socket, equipped with safety fuse that can be checked in case of failure.
- The provided power supply unit is specifically designed for use with the **Port A Suction**. Never use this power supply unit with other devices or for uses different from the one described in this manual and never use the **Port A Suction** with other power supply units.

The appliance contains a sealed lead-acid storage battery. Used batteries must be disposed of in compliance with the prevailing legislation in respect of toxic/hazardous waste management. Contact an authorised service centre.

## INSTRUCTIONS FOR USE

Before each use, the accessories should be carefully checked so as to make sure that there is no dust, deposits, lumps or liquid substances inside the connection tubes, the collection jar and the relevant lid. Accurately clean, disinfect or sterilise these parts strictly following the instructions for **CLEANING AND DISINFECTION**.

### 1. Operation with internal battery:

- 1.1. The battery supplied with the appliance is partly charged; it should be recharged before use. For recharging follow the instructions under **2.3.1**.
- 1.2. Operate the unit by turning the switch on (3); the green led "PUMP ON" (6A) will turn on. Do not forget that the autonomy with fully charged batteries is approx. 30 minutes.
- 1.3. After using the appliance, always shift the switch (3) to 0 (OFF) even when the batteries are low. If the red indicator light "LOW BATTERY" (6B) turns on during use, the appliance has exhausted its autonomy of operation. Turn the appliance off immediately and recharge.

### 2. Operation and recharging through car cigarette-lighter cord (12V DC) or multivoltage power unit:

#### 2.1. Cigarette-lighter cord (12V DC)

- 2.1.1. Connect the external socket (2) of the appliance to the cigarette-lighter socket by means of the cigarette-lighter cord.

*Check the car or boat battery charge level before using the cigarette-lighter cord.*

- 2.1.2. Operate the unit by turning the switch on (3); the green led "PUMP ON" (6A) and the green led "EXT. POWER" (6D) turn on to confirm operation. Bear in mind that the appliance is intended for intermittent use 30 min ON/30 min OFF.

#### 2.2. Multivoltage power supply unit (switching)

- 2.2.1. Connect the power lead plug to the external socket (2) of the appliance. If the power lead plug is different from the electrical outlet, contact your nearest dealer or authorized service centre.

- 2.2.2. Follow the instructions under **2.1.2**

#### 2.3. Internal battery recharge through car cigarette-lighter cord or multivoltage power unit:

*Check the car or boat battery charge level before using the cigarette-lighter cord.*

- 2.3.1. The batteries are charged by leaving either the cigarette-lighter cord or the multivoltage power supply unit connected to the external socket (2) of the appliance. The green led "EXT. POWER" (6D) and the yellow led "CHARGING" (6C) turn on to confirm operation.

For first time use recharge the appliance for 24 hours; later on 8 to 12 hours are sufficient according to the battery charge left. If required, the appliance can be used before the charging is finished provided that, once the application is completed, the recharging operation is continued.

### 3. Instructions for tracheotomy patients:

- 3.1. Connect the accessories as shown in the drawing on page 2.
- 3.2. Turn on the appliance following the relevant operating instructions. *Always operate the unit on a flat surface; this will allow the correct operation of the float shut-off thus preventing entrance of liquid in the suction pump.*
- 3.3. The desired level of vacuum (bar) can be set by using the vacuum regulation knob (4). A higher vacuum level is obtained by turning the knob clockwise; a lower vacuum level is obtained by turning the knob counter-clockwise. These values can be read on the vacuum gauge (5). In order to make suction faster and cleaning operations simpler, fill the jar (16) with approximately 400 ml of water.
- 3.4. Put your finger on the hole of the manual suction regulator (17) and start the treatment by making alternate upward and downward movements on the hole with your finger.
- 3.5. When the treatment is finished, turn off the unit.
- 3.6. Empty and clean the jar and the connection tubes.

## **CLEANING AND DISINFECTION**

- Disconnect the suction catheter (18), the manual suction regulator (17) and the tube (15) from the collection jar (16). Disconnect the tubes (8/10) both from the jar and the antibacterial filter (9) and empty the unit (into the WC at home and into the biological material container in a hospital) and clean the jar. In order to clean the accessories immediately after use, it is sufficient to rinse the individual parts under running water. All accessories can be disinfected or sterilised choosing one of these methods:
  - a) by boiling (5-10 mins.); this methods cannot be used for the connection tubes
  - b) by active-chlorine-based chemical disinfectant which is commonly available in drugstores and pharmacies
  - c) by cold chemical sterilizers (peracetic acid based).

**NOTE: should chemical disinfectants and/or sterilizers be used, strictly follow the instructions provided by the manufacturer.**

- The suction catheter and the manual suction regulator are **disposable** sterile products and should be replaced after each application.
- The **disposable** antibacterial filter should be replaced within 24 hours after its first use.
- Never immerse or wash the unit under running water; wipe clean the unit external body only with a damp cloth using a mild detergent. (Do not use abrasive cleaners).
- Reassemble the collection jar and the connection tubes according to the instructions in the section "**INSTRUCTIONS FOR USE**".

## **ACCESSORIES - TECHNICAL FEATURES**

- **Disposable** antibacterial filter
- 1000 ml Collection jar
- Connection tube: Ø 8.5 x 13.5 mm, 80 mm long
- Connection tube: Ø 8.5 x 13.5 mm, 160 mm long
- Connection tube: Ø 8.5 x 13.5 mm, 1300 mm long
- **Disposable** sterile suction catheter: CH 18 Ø 6 x 4.4 mm, 530 mm long
- **Disposable** sterile manual suction regulator

### **Microbial contamination**

The accessories, collection jar and connection tubes are for single person use only. If used by more than one person cross infections may occur. Consult your physician if in any doubt.

### **Cross reactions**

The materials used in manufacture of the unit that come into contact with body-secretions are thermoplastic polymers. These polymers have been selected due to their excellent chemical stability and resistance (PP, PC, SI). The possibility of a chemical interaction, however, should not be excluded. The following precautions are therefore recommended:

- a) Prolonged contact between liquid and collection jar or tubes should be avoided; always wash, disinfect or sterilise these parts immediately after use.
- b) If unexpected situations occur, for example, the accessories crack or become soft, stop treatment immediately and substitute the accessories used. Contact an authorised dealer for service specifying how suctioning was being performed.

**NOTE: use only FLAEM's original accessories**

## **WARRANTY**

The unit is under warranty from the dealer that sold you the unit.

## TECHNICAL FEATURES

### UNIT

Reciprocating twin-cylinder piston compressor, designed for use without lubrication. The housing is made of metal painted with epoxy powders.

Voltage:	12 V
Power:	3 A
Adjustable vacuum level:	from -0.16 to -0.81 bar approx
Max air flow:	20 l/min approx
Dimensions:	37 x 10 x 24 cm
Weight:	5.6 Kg
Sound level (at 1 m):	65 dB (A) approx
Intermittent use:	30 mins. ON/30 mins. OFF (when operating through cigarette-lighter cord or multivoltage power supply unit)

Operating requirements:	temperature min. 0°C; max 40°C air humidity min. 10%; max 95%
Storage requirements:	temperature min. -15°C; max 50°C air humidity min. 10%; max 95%

Operating/storage atmospheric pressure:	min. 690 hPa; max. 1060 hPa
Internal battery:	2x6V DC 3Ah Lead air-tight
Battery autonomy:	30 mins. approx.

### POWER LEAD FROM CIGARETTE LIGHTER CODE 07261A0

In case of breakage of fuse, replace with a 5A - 250V approved fuse, rapid, having dimensions Ø 6.3 x 30 mm, unscrewing the end of the plug that must be fitted in the cigarette-lighter socket.

### MULTIVOLTAGE POWER SUPPLY UNIT - SWITCHING CODE 09818

(Model: SA165A-1250U-3)  
Primary: 100-240V ~ 50/60Hz  
Secondary: 12V   
Dimensions: 12 x 6 x 4 cm  
Weight: 300 g

In compliance with Directive 93/42/EEC 0051

**Disposal:** In compliance with Directive 2002/96/EC, the symbol shown on the appliance to be disposed of indicates that this is considered as waste and must therefore undergo "sorted waste collection". The user must therefore take (or have taken) the above waste to a pre-sorted waste collection centre set up by the local authorities, or else give it back to the dealer when purchasing a new appliance of the same type. Pre-sorted waste collection and the subsequent treatment, recovery and disposal operations favour the production of appliances made of recycled materials and limit the negative effects of any incorrect waste management on the environment and public health. The unlawful disposal of the product by the user could result in administrative fines as provided by the laws transposing Directive 2002/96/EC of the European member state in which the product is disposed of.

## SYMBOLS

Warning: follow the "Instructions for use" Direct current 0051 Medical Marking ref. 93/42/EEC Directive

Switch "OFF" (or battery recharging) Switch "ON" Do not use the unit while having a bath or a shower

Clockwise - Higher vacuum value  
 Counterclockwise - Lower vacuum value

## EXHAUST

Classe II unit

Alternating current

Type B unit

Sterilization by means of ethylene oxide

Disposable

Latex Free

### Electromagnetic compatibility

This suction unit is designed to meet the current requirements for electromagnetic compatibility. However, if you suspect that the unit is interfering with the operation of your TV, radio or other electric appliance, try relocating the unit until the interference ceases or plugging the unit into a different power socket during the battery recharge mode.

The Manufacturer, the Retailer, the Importer are responsible for safety, reliability and performance of the unit only if:

- the unit is used following the instructions for use;
- the electrical system of the facility where the unit is used is in accordance with IEC Standards;
- assembly, additions, re-calibrations, changes or repairs are carried out by authorised personnel.

The Manufacturer reserves the right to make technical changes to this product without prior notice.

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